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REL EYEWEAR  
Italian Design Eyewear

## USE INSTRUCTIONS (EN)

Read and keep the following instructions which will help keep the device in perfect working order.

### **Description**

The optical frame is a class I medical device whose intended use is the adaptation to prescription glasses in association with ophthalmic lenses or other suitable filters.

### **Warnings**

- It is not personal protective equipment: it is not intended for eye or face protection.
- It is not a complete custom-made pair of glasses.
- It's not sunglasses.
- It is not intended for use as is.
- The frame is not intended for use in extreme conditions that could result in damage to the device and risk to the wearer.

### **Indications for correct use**

- The frame must be handled, adjusted, modified within the limits of these instructions for use, only by an optician or other qualified, competent personnel with the necessary experience for these activities.
- The operations described above are part of the art and profession of the optician, must be acquired with appropriate training courses and experience, and cannot be part of this instruction manual. Only correct manual operation can guarantee the suitability of the device and its correct functionality. The device is designed and manufactured to properly respond to correct optical handling.
- Do not flex the components of the frame beyond the reasonable limit, or in any case beyond what is necessary for a normal fit on a face of a size compatible with the frame itself.
- If the sides have steel terminals covered with silicone or other flexible rubber terminal covers, heating the component is not required to change the curvature or shape. In this case, limit the modification to the flexible parts.
- The PA12 nylon temple and front components can be adjusted and modified to the extent permitted by the thickness of the material, after heating.
- Any adjustment, modification or alteration by unqualified personnel can lead to damage to the device, with consequent safety risks, including future ones, during use.
- The non-optical and non-expert end user must not independently carry out the adjustment operations described above or other invasive actions towards the device: contact the optician.
- The frame without lenses or equipped with presentation lenses must not be supplied and used as such, but can be worn only for testing on the face at the time of evaluation.
- Even if fitted with lenses, the frame must not be supplied for use to the end user unless provided with the appropriate adapted device documentation issued by the adapter.
- The device is designed and manufactured to be mounted with ophthalmic lenses or other suitable filters

**REL Eyewear – Riccardo Cervio**

Via Palestro 126, 27038 Robbio (PV) - P.ta IVA 02670740188 - CF CRVRCR88M09G842P  
mail: [info@releyewear.it](mailto:info@releyewear.it) - web: [www.releyewear.it](http://www.releyewear.it)

and to fit adequately on the wearer's face. Only the evaluation of expert and competent personnel can effectively evaluate the compatibility with the individual wearer's face, making it in all respects an adapted device. If the device does not fit properly it must not be supplied and used.

- The device is designed and manufactured to be used in connection with ophthalmic lenses or other suitable filters, even for a prolonged period of time quantifiable in several years, if these instructions for use, maintenance and cleaning are correctly followed.
- Put on the frame and remove it from the face by holding both temples.
- Do not force the movement of the sides beyond the maximum opening or closing limit, whether they are flexible or not.
- If the sides feel strangely stiff, do not force them to move but lubricate the moving mechanisms.

### ***Maintenance***

Maintenance activities must be carried out both by the optician and by the end user, who must contact the optician for any information or assistance.

- Periodically check the setting of the frame.
- Periodically check the correct tightening of the screws.
- Periodically inspect the device, looking for cracks, deformations, surface alterations of the material.
- Contact your optician or other qualified personnel for the operations described above or whenever you suspect or highlight an anomaly in the structure or functionality of the device.

### ***Cleaning instructions***

- Use only recommended products and neutral liquid detergents to clean the device and dry it with appropriate fabric.
- The recommended products are water at room temperature and neutral soap, preferably with a specific composition for frames and eyeglasses.
- Although the materials used are of high quality and resistant to chemical agents, the use of alcohol or other aggressive agents is strongly discouraged.
- The use of solvents or chemical agents known to interact aggressively with organic materials, such as acetone, is strongly discouraged.
- In the absence of the recommended products, use water.

**REL Eyewear – Riccardo Cervio**

Via Palestro 126, 27038 Robbio (PV) - P.ta IVA 02670740188 - CF CRVRCR88M09G842P  
mail: [info@releyewear.it](mailto:info@releyewear.it) - web: [www.releyewear.it](http://www.releyewear.it)